



Applying the HIPSTER™

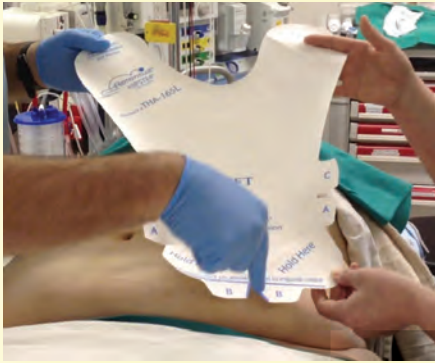

1. Orient line about 5 cm above anticipated incision. The inverted V is placed directly above the top of incision.

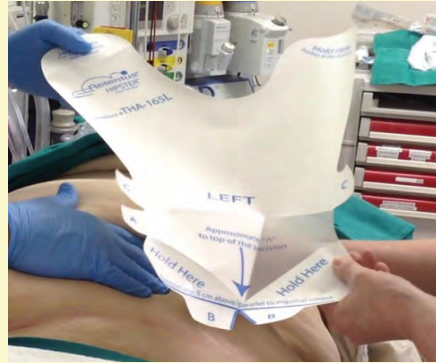

2. Manually retract pannus & remove A tab.



3. With pannus still retracted, place panel A on patient & smooth down.



4. Smooth panels down as you remove B tabs.




5. Once B tabs are removed, retro fold and release manual retraction.



6. Holding device in tension, with a hand firmly on the A panel, pull C tabs.



7. Holding device 45° over patient, apply retractor to patient.

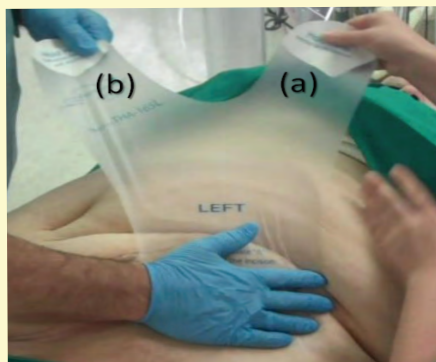

8. Firmly hold down A Panel, push up to retract pannus. Pull & stretch HIPSTER cephalad toward (a) xiphoid & (b) cross-midline under opposite breast.



9. Attach (a) at xiphoid



10. Attached (b) under opposite breast.



11. Completely smooth down to skin.



12. Completed retraction